




In-app Practice Alerts

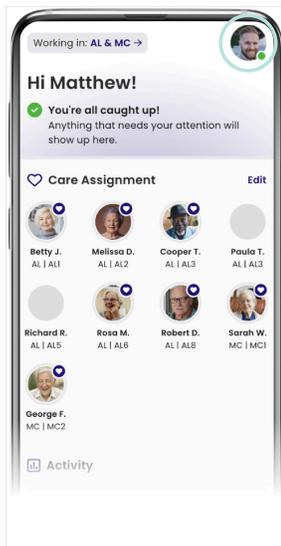

Tap on your **profile icon** in the top right corner

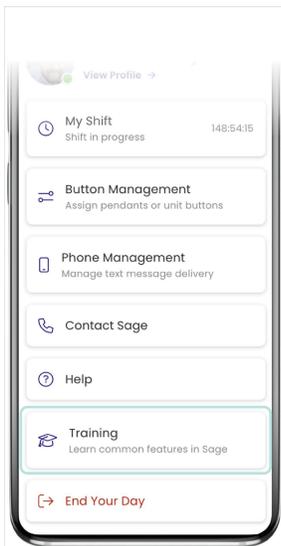

Scroll down and tap onto the **'Training'** menu button

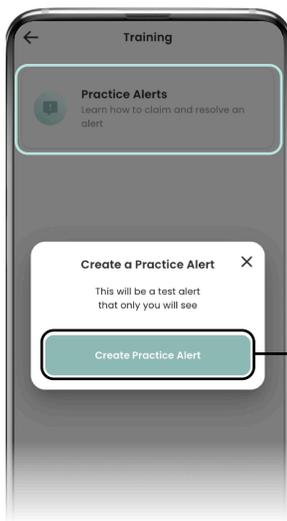


Tap on the **'Practice Alerts'** option to begin

A unique alert made just for you!


When a team member creates a practice alert only they can see it.

This means they can practice responding to alerts as many times as needed without distracting other team members.



A Practice Alert will be created

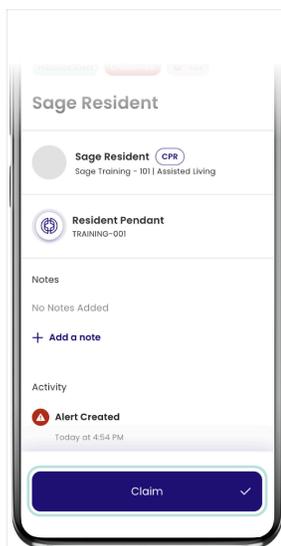


Tap **'Claim'** to simulate your move to assist the resident

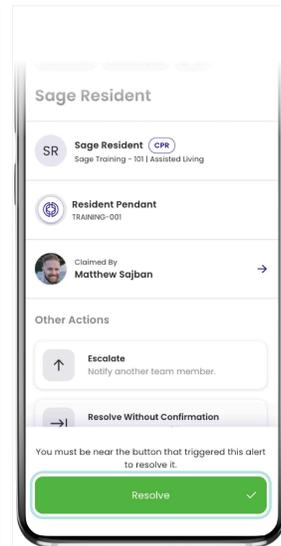

Tap **'Resolve'** to simulate completion of resident care

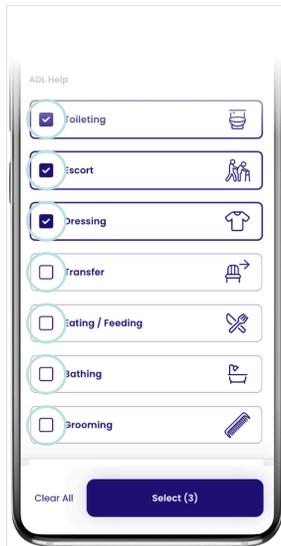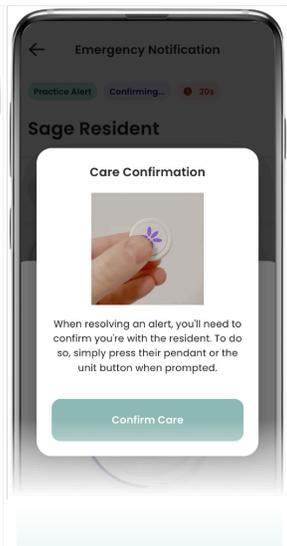

Review and select care you might provide to a resident

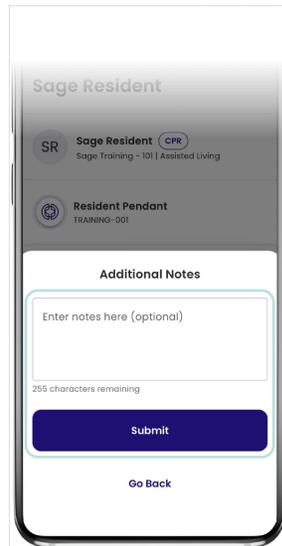

Enter a note (optional) and tap **'Submit'** to clear the alert



Press the Right Button

For real life alerts you will need to press either the resident's pendant or the unit button. You will notice resident pendants have a photo or the resident's initials, unit buttons have an image of a house.



Pendant



Unit Button