





# Prepare Your Room for Fall Management

Keeping the floor area clear will help prevent falls as well as make the system more effective. Here are suggestions for proper room setup.






Sage Detect







## Key Areas to Keep Clear

-  The floor beside the bed
  -  The walking path from the bed to the bathroom
  -  Any open floor space in the room
-  For the system to work properly, the floor must remain visible to the camera at all times.

## Please DON'T

-  Leave boxes, bins, or suitcases on the floor
-  Leave piles of clothing, blankets, or bags on the floor
-  Place furniture in the middle of the room covering the floor
-  Block the path from bed to bathroom with bulky furniture
-  Place large pieces of furniture directly under the detect hardware

## Please DO

-  Use small bedside mats that don't cover walking paths
-  Place personal items on shelves, dressers, or nightstands
-  Arrange furniture against the walls without the camera where possible
-  Decorate with photos, plants, and items that make your new home uniquely yours!

## Questions?

**Please speak with your care team.** They can help arrange the room for safety and comfort.