


Introducing Sage & Sage Detect

We are delighted to announce that Sage and Sage Detect, a new state-of-the-art Nurse Call and Fall Management System, is coming to our community!

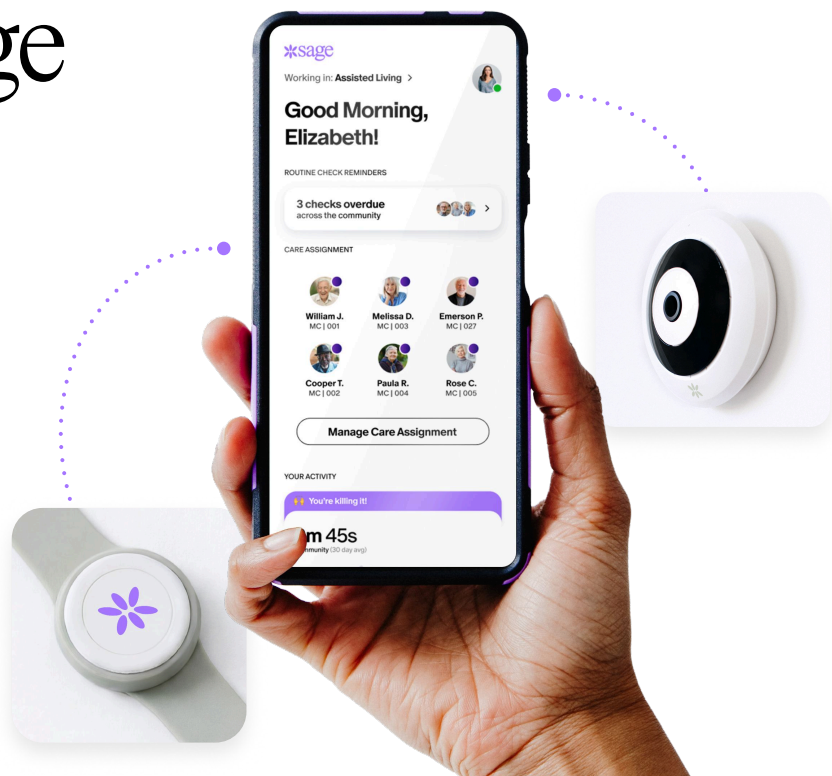

How does Sage work?

Residents will receive Sage pendants and call buttons will be installed in each bedroom and bathroom. When a Sage device is pressed, it alerts the care team that assistance is needed.

What if I fall and can't press for help?

That's where Sage Detect comes in. With Sage Detect, an alert is sent out if it detects a person on the ground even if a Sage button is not able to be pressed by the individual.

Does this change my daily routine?

No! Sage nurse call will simply replace the current pendants and pull cords.

Sage Detect is optional. Each room will be equipped with a camera, however it will have a cover and will not be activated until the resident chooses to opt-in.

How does Sage Detect work?

Sage Detect uses a camera, installed in the resident unit, to trigger an alert if the system identifies a potential fall has occurred - reducing time on the ground or possible ER visits. The system will send a video clip of the event to the community clinician to help triage care.

How is my privacy protected?

Sage Detect will only be activated in those units where the resident has signed the consent form.

Sage Detect does not record audio or provide live video access. The camera is only used to detect a potential fall - the videos of these events are shared only with the pre-authorized individuals.

Anyone who opts in to Detect can opt-out and have the camera de-activated and covered at any time.

Want to learn more?

Please speak with your care team. They will be more than happy to answer any additional questions you may have.