





# Prepare Your Room for Fall Management






Keeping the floor area clear will help prevent falls as well as make the system more effective. Here are suggestions for proper room setup.







## Key Areas to Keep Clear

-  **The floor beside the bed**
  -  **The walking path from the bed to the bathroom**
  -  **Any open floor space in the room**
-  For the system to work properly, the floor must remain visible to the camera at all times.

## Please DON'T

-  Leave boxes, bins, or suitcases on the floor
-  Leave piles of clothing, blankets, or bags on the floor
-  Place furniture in the middle of the room covering the floor
-  Block the path from bed to bathroom with bulky furniture
-  Place large pieces of furniture directly under the detect hardware

## Please DO

-  Use small bedside mats that don't cover walking paths
-  Place personal items on shelves, dressers, or nightstands
-  Arrange furniture against the walls without the camera where possible
-  Decorate with photos, plants, and items that make your new home uniquely yours!

## Questions?

**Please speak with your care team.** They can help arrange the room for safety and comfort.